
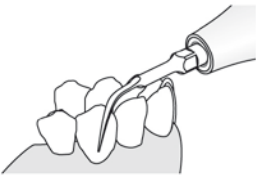
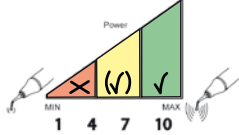
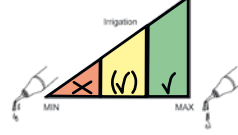
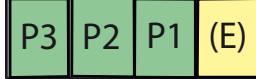




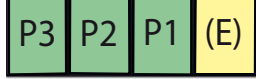

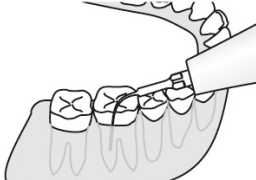

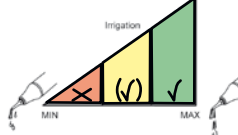
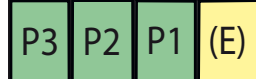



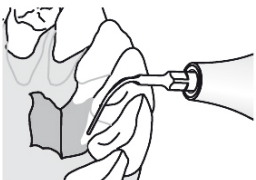

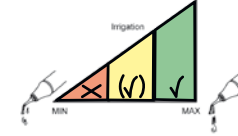
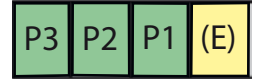
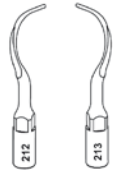



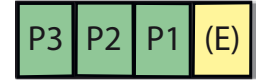



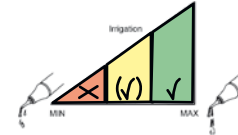
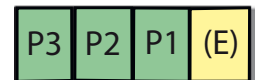
Adjustment Recommendation of PiezoLED

PIEZOscaler Tips

Product code	Therapeutic Indication	Power Setting	Spray Water Volume	Mode Adjustment*
 Scaler 201				
 Scaler 202				
 Scaler 203				

* Only applicable for ESTETICA - treatment units, PiezoLED for Primus 1058 is bound to P3

PIEZO Paro Tips



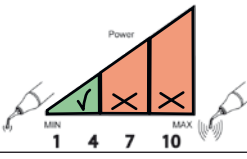
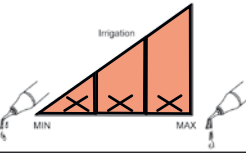

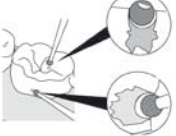
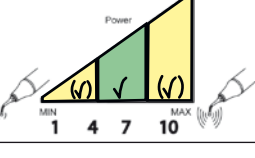
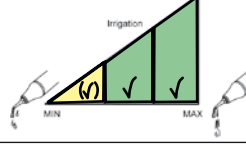

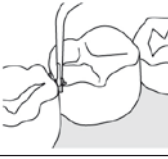
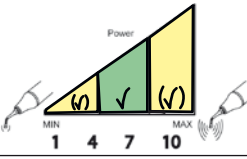
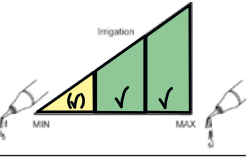

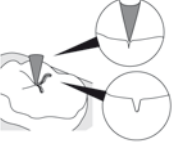
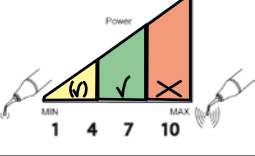
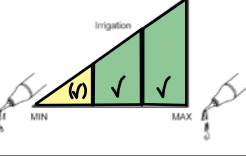
Product code	Therapeutic Indication	Power Setting	Spray Water Volume	Mode Adjustment
 Paro 210 + 211				
 Paro 212 + 213				
 Paro 214				

█ recommended
 █ possible
 █ not recommended






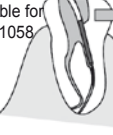

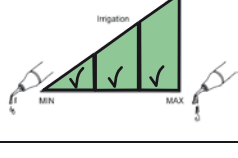

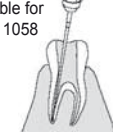

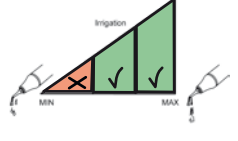


KaVo PiezoLED® Ultrasonic-Scaler


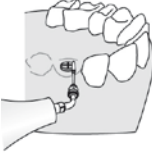

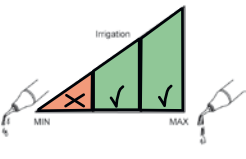
PIEZO Prep Tips

Product code	Therapeutic Indication	Power Setting	Spray Water Volume	Mode Adjustment
 Cem 225				(P3) (P2) (P1) E
 Prep 226				P3 P2 P1 (E)
 Prep 227 + 228				P3 P2 P1 (E)
 Prep 229				P3 P2 P1 (E)

PIEZO Endo Tips

Product code	Therapeutic Indication	Power Setting	Spray Water Volume	Mode Adjustment
 Endo 220				P3 P2 P1 (E)
 Endo 221*	Tip not applicable for Primus1058 			P3 P2 P1 E
 Endo 222* with files	Tip not applicable for Primus 1058 			P3 P2 P1 E

PIEZO Implant Tips

Product code	Therapeutic Indication	Power Setting	Spray Water Volume	Mode Adjustment
 Implant Set				P3 P2 P1 (E)

■ recommended
 ■ possible
 ■ not recommended

* Only applicable for PiezoLED with ESTETICA treatment units



KaVo. Dental Excellence.