


1. Move Ceph Sensor to Ceph Sensor Holder

Note: If your system includes two sensors, skip this step

2. On The Touchscreen

A. Select the Ceph Tab

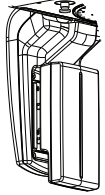

B. Select Patient Size



Large
73kV



Medium
70kV



Small
65kV

C. Select the desired Ceph projection

**3. On The Control Panel**

Press the *Patient In* button and adjust the unit height


**4. Patient Positioning**

A. Unlock the lever and turn the ear rods to the Lateral or PA projection position. Lock the position.

B. Guide the patient to the unit and instruct them to stand as straight and tall as possible.



C. Slide the ear rods towards patient's ears.

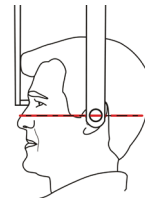

D. Turn on the lasers, adjust the unit height to get the Frankfort plane laser passing over orbitale and porion.

E. Tilt the nasion support down and slide it towards patient's nasion.

5. Instruct Patient

A. Ask the patient to swallow, bite down and close their lips.

B. Remain still during scan.

6. Take The Scan

Press and hold the exposure button until the rotating unit stops.

